

Tips for Universal Masking

- Mask is secured over nose and mouth



- Correct way to store ear loop mask when not in use- exterior of mask is facing DOWN



- Correct way to store tie mask when not in use- Exterior of mask is facing DOWN and ties are placed away from the inside of the mask



Tips for Universal Masking

- Inappropriate wear of ear loop mask- mask should **not** be pulled under the mouth or chin



- Inappropriate wear of tie mask- mask should **not** hang from lower ties



- Incorrect way to place mask when not in use- exterior of mask should **not** be up and ties should **not** touch the interior of the mask



OR

